
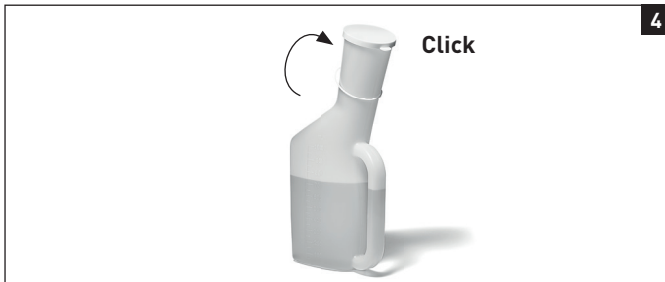
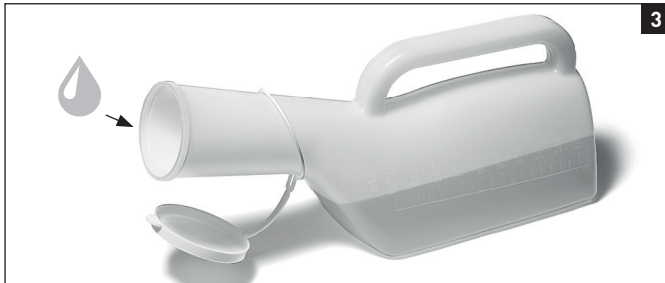
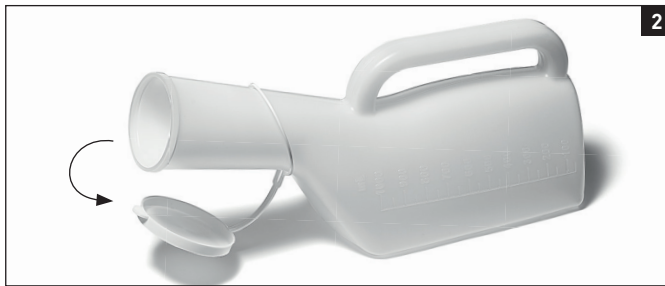
Urinal men

DE: Urinal - Männer / ES: Orinal - masculino / FR: Urinal - homme / IT: Orinale - maschile / NL: Urinaal - man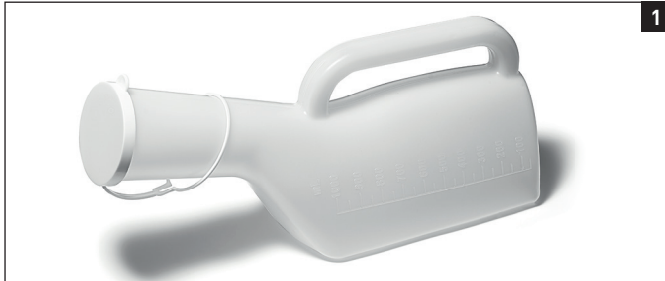

EN User manual **Urinal** men

Be sure to read all information and directions before use! Keep this manual for future use.

Use: If you need to urinate, hold the urinal in one hand and place your penis in the urinal with your other hand. Try to lead the penis as far as possible into the urinal in order to decrease the chance of spilling.

Sitting or lying down: An advantage of this male urinal is that you do not need to stand up in order to use it. You can use it while lying in bed or on the couch. Of course, you can also use it while sitting upright. This makes the urinal a convenient tool if you have trouble standing up.

Closing: The urinal can be closed with a lid. Keep the lid closed before and after use in order to prevent odours and spilling. The lid is made of sturdy plastic.

Maintenance: Always thoroughly rinse out the urinal with hot water and use an all-purpose cleaner to clean the urinal. With the Vitality urinal brush you can clean the product even more easily and thoroughly. Safe manual cleaning up to 50 °C

Place the urinal in a convenient spot next to your bed. Special urinal holders are also available.

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MIXED.Industries B.V.