
Quick mobile rail 49 cm / 19.3 inch

EN: Quick mobile rail - 49 cm / 19.3 inch / DE: Wandbügel mit Schnellbefestigung - 49 cm / 19.3 inch / ES: Asidero con ventosas - 49 cm / 19.3 inch / FR: Barre d'appui Quick mobile rail- 49 cm / 19.3 inch / IT: Maniglia mobile Quick Mobile - 49 cm / 19.3 inch / NL: Wandbeugel Quick Mobile Rail - 49 cm / 19.3 inch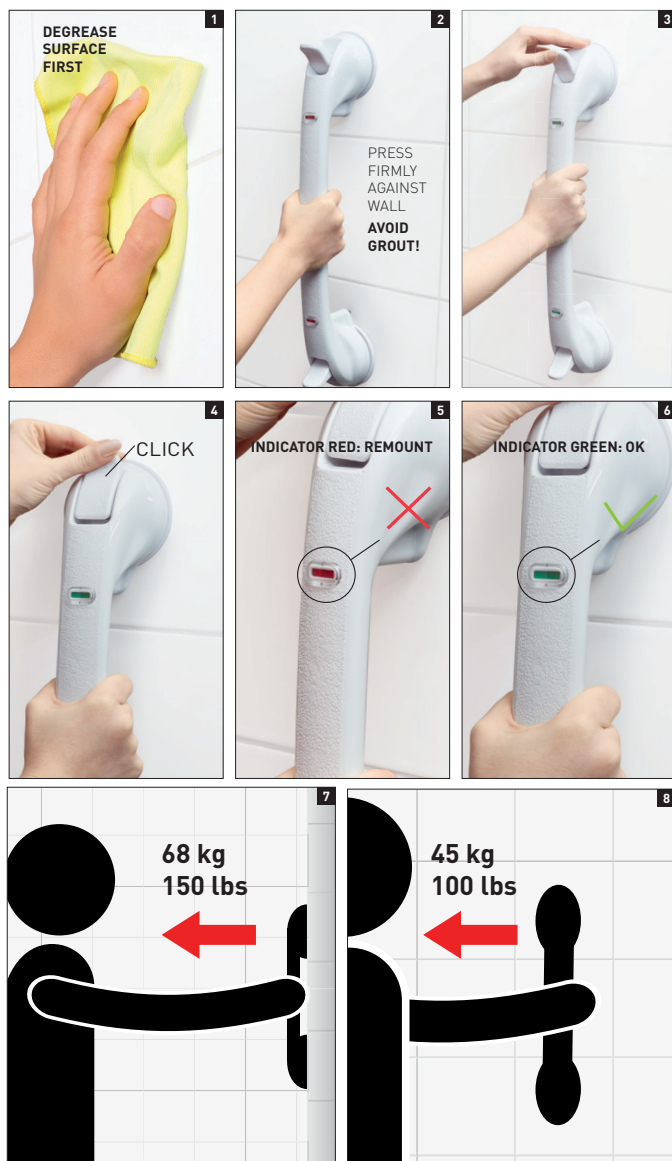

EN User manual Quick mobile rail 49 cm / 19.3 inch

Be sure to read all information and directions before use!
Keep this manual for future use.

- 1 Degrease the surface before attaching the Quick mobile rail.
- 2 Attach the Quick mobile rail on flat surfaces only, avoid grouts.
- 3 Push the Quick mobile rail firmly, till the indicator turns green.
- 4 Attach the Quick mobile rail correctly by snapping the two handles. You should feel resistance as the suction cup tightens. Before each useage of the Quick mobile rail check the indicator.
- 5 When the indicator indicate red, reattach the Quick mobile rail.
- 6 When the indicator indicate green, it is safe to use the Quick mobile rail.
- 7 Maximal vertical supporting weight 68 kg / 150 lbs.
- 8 Maximal lateral supporting weight 45 kg / 100 lbs.

WARNING: the Quick Mobile Rail can't be used to support full body weight. The Quick Mobile Rail is designed to assist your natural motion when stepping into or out of a shower. Maximal supporting weight: Vertical: 68 kg / 150 lbs, Lateral: 45 kg / 100 lbs. Only use on smooth and non-porous surfaces: tile, glass, porcelain or fiberglass. Not for use on wall board, wood, painted or papered surfaces! Do not attach to moving surfaces. Do not use a sharp instrument by removing the Quick Mobile Rail. This may damage the suction cup. Always keep the handles up until next use.

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